

Clearblue®

Pregnancy Test DIGITAL

with Smart Countdown


- Clearblue® Digital is an early result pregnancy test which can be used up to 4 days before your expected period (which is 5 days sooner than the missed period).
- Clearblue® Digital is more than 99% accurate when used from the day you expect your period to start*.

1 TAKE THE TEST

- Please read these instructions carefully before taking a test.
- When you are ready to test, remove the test stick from the foil wrapper and take off the cap. Use the test stick right away.

IMPORTANT

- Throughout testing **never** hold the test stick with the absorbent tip pointing upwards.

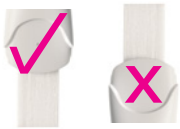

Do

Don't

EITHER

TEST IN YOUR URINE STREAM

- Place just the absorbent tip pointing downward in your urine stream for **5 seconds only**.
- Take care not to get the rest of the test stick wet.
- Keeping the absorbent tip pointing downward, replace the cap and lay the test stick flat.



Absorbent Tip

OR

COLLECT A SAMPLE OF YOUR URINE IN A CLEAN, DRY CONTAINER

- Place just the absorbent tip in the urine, as shown, for **20 seconds only**.
- Keeping the absorbent tip pointing downward, replace the cap and lay the test stick flat.



2 Clearblue® SMART COUNTDOWN

- The Smart Countdown will be shown on the display so you know the test is working and each segment will appear in sequence (flashing), to show the test is progressing.



Smart Countdown

- If the Smart Countdown does not appear after you have taken the test, refer to Q5.

3 READ YOUR RESULT

- Once the Smart Countdown is complete it will disappear from the display and your result will be shown.
- This will take a maximum of **3 minutes** but some results may be shown in as little as 1 minute.
- A 'Pregnant' result will remain on the display for up to 6 months.
- A 'Not Pregnant' result will remain on the display for approximately 24 hours.



If your result is 'Pregnant' see Q9.



If your result is 'Not Pregnant' see Q10.

Questions and Answers - Please read the section on the opposite page which answers the most frequently asked questions.

Questions and Answers

1 How accurate is Clearblue® Digital Pregnancy Test?

Clearblue® Digital Pregnancy Test is more than 99% accurate in laboratory tests, when used from the day you expect your period to start*. If you are testing earlier than the day of your expected period the accuracy of Clearblue® Digital will be lower (see Q3).

2 How does Clearblue® Digital Pregnancy Test work?

When you are pregnant your body produces the pregnancy hormone hCG (human Chorionic Gonadotrophin). The amount of hCG in your body increases in the early stages of pregnancy. Clearblue® Digital Pregnancy Test can detect tiny amounts of this hormone in your urine.

3 When can I test with Clearblue® Digital Pregnancy Test?

Clearblue® recommends that you test from the day your period is expected. However, the pregnancy hormone increases rapidly in early pregnancy and this test can be used to test up to 4 days before you expect your period (which is 5 days sooner than the missed period). If you test before your expected period and get a 'Not Pregnant' result, there is still a chance that you may be pregnant. See the table below for results from clinical testing of Clearblue® Digital with early pregnancy samples.

Testing Early				
Number of days before the expected period	-1	-2	-3	-4
% of samples from pregnant women giving a 'Pregnant' result.	95%	90%	82%	51%

- To figure out when your period is due, calculate your usual cycle length by counting the number of days from the first day of your period until the day before your next period starts.
- If you have irregular cycles, you should allow for your longest cycle in recent months before testing.
- If you have no idea when your period is due, we recommend testing at least 19 days after the last time you had unprotected sex.
- If you are testing before you expect your period to start, we recommend you test with the first urine of the day; otherwise you can test whenever you next need to pass urine. Avoid excess fluid intake before testing.

If the result is 'Not Pregnant' and pregnancy is still suspected, you should wait at least 3 days before testing again. See Q10.

4 Can any medication or medical conditions affect the results?

- Always read the manufacturers' instructions for any medication you are taking before conducting the test.
- Fertility drugs containing hCG (such as Pregnyl® and Profasi®)** can give misleading results. These fertility drugs are usually given by injection, and testing too soon after administration may give a false 'Pregnant' result.
- Other fertility therapies (such as Clomid®)***, painkillers and hormonal contraceptives (e.g. birth control pills) are unlikely to affect the result.
- If you have recently stopped taking hormonal contraception or are using fertility therapies like Clomid®, your periods may be irregular, leading you to test too soon.
- If you have recently been pregnant (even if not carried to full-term) you may get a false 'Pregnant' result.
- If you are in or approaching menopause you may obtain a false positive result even though you are not pregnant. In a study of women tested with different lots of Clearblue®, an average of 0.3% of results were positive, with a range from 0% to 1% for different lots. All false positive results were from women aged over 50 years.
- Ectopic pregnancy and ovarian cysts may give misleading results.
- If you have a 'Pregnant' result, it is possible that you may later find that you are not pregnant due to a natural loss of the pregnancy which can occur in the early stages.

If you get unexpected results you should discuss them with your doctor.

5 I have taken the test but the Smart Countdown has not appeared what does this mean?

The test has not worked correctly. Please contact the Clearblue® Helpline toll free 1-800-321-3279 for further information.

6 The Smart Countdown counted down much faster than I expected, what does this mean?

Some results can be displayed in as little as 1 minute. In such cases the Smart Countdown segments will appear more rapidly.

7 The Smart Countdown started but did not finish and no result appeared on the display, or an error is now displayed. What does this mean?

The test has not worked correctly; see 'Test Errors'.

8 I have used the test but no result has appeared on the display. What does this mean?

Your result should appear on the display within 3 minutes. If no result appears please see 'Test Errors'.

9 My test says I'm Pregnant. What should I do?

If your result is 'Pregnant' you should see your doctor who will advise you on what steps you should take next.

10 My test says I'm Not Pregnant. What should I do?

You may not be pregnant or the level of pregnancy hormone may not yet be high enough to be detected, or you may have miscalculated the day your period is due. If you tested early, test again when your period is due. If your period is overdue, test again in 3 days time. If this test gives you a 'Not Pregnant' result and your period has not started, see your doctor.

Test Errors

Book Error

An error has occurred during testing. It may be that:

- The absorbent tip was not kept pointing downward or the test was not laid flat after urine was applied.
- Too much or too little urine was applied. You should test again using a new test. Take care to follow the instructions.

Blank Error

If the display is blank after you have used the test please contact the Clearblue® Helpline toll free 1-800-321-3279 for further information.

* 99% accurate at detecting the pregnancy hormone. However, some pregnant women may not have detectable amounts of pregnancy hormone in their urine on the first day of the expected period or may have miscalculated the first day of their period.

Clearblue® Digital Pregnancy Test is designed for easy use at home. However if you have questions about the test please call the Clearblue® Helpline Monday-Friday 8:30 a.m. - 5:00 p.m. Eastern Time. Toll-free. 1-800-321-3279. Please have the lot numbers **LOT** from both the outer and inner packaging. Calls are recorded for training and quality control.

Read and share product reviews at: www.clearblueeasy.com [facebook.com/clearblue](https://www.facebook.com/clearblue) twitter.com/clearblue

Home pregnancy test. Use once. For in vitro diagnostic use only. Not for internal use. Keep out of the reach of children. Store at 36°F-86°F (2°C-30°C).

Allow to reach room temperature for 30 minutes prior to testing if stored refrigerated. Do not use if the foil wrapper containing the test stick is damaged. Do not use the test stick after expiration date. This product incorporates a battery. Dispose of in accordance with local regulations.

Made in PRC. Packaged in the USA. Distributed by Procter & Gamble, Cincinnati, OH 45202.

Clearblue is a registered trademark of SPD, Swiss Precision Diagnostics GmbH. © 2013 SPD, Swiss Precision Diagnostics GmbH. All rights reserved.

Protected by U.S. Patent Nos. 7,315,378; 7,317,532 and 6,187,598. Other rights granted and pending.

Pregnyl® is a registered trademark of ORGANON USA, Inc. Profasi® is a registered trademark of Ares Trading S.A. P/N 505103-10 07/2013. *Clomid® is a registered trademark of Sanofi-Aventis

This device complies with part 15 of the FCC rules. Operation is subject to the following conditions:

(1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communication. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to television

or radio reception, which can be determined by turning the equipment on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase separation between equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Contact the dealer or an experienced radio/TV technician for help.

The US Surgeon General advises pregnant women to abstain from alcohol consumption as this may lead to harmful effects on the baby such as Fetal Alcohol Syndrome Disorder (FASD). More information is available at www.surgeongeneral.gov